


AKED.bzh

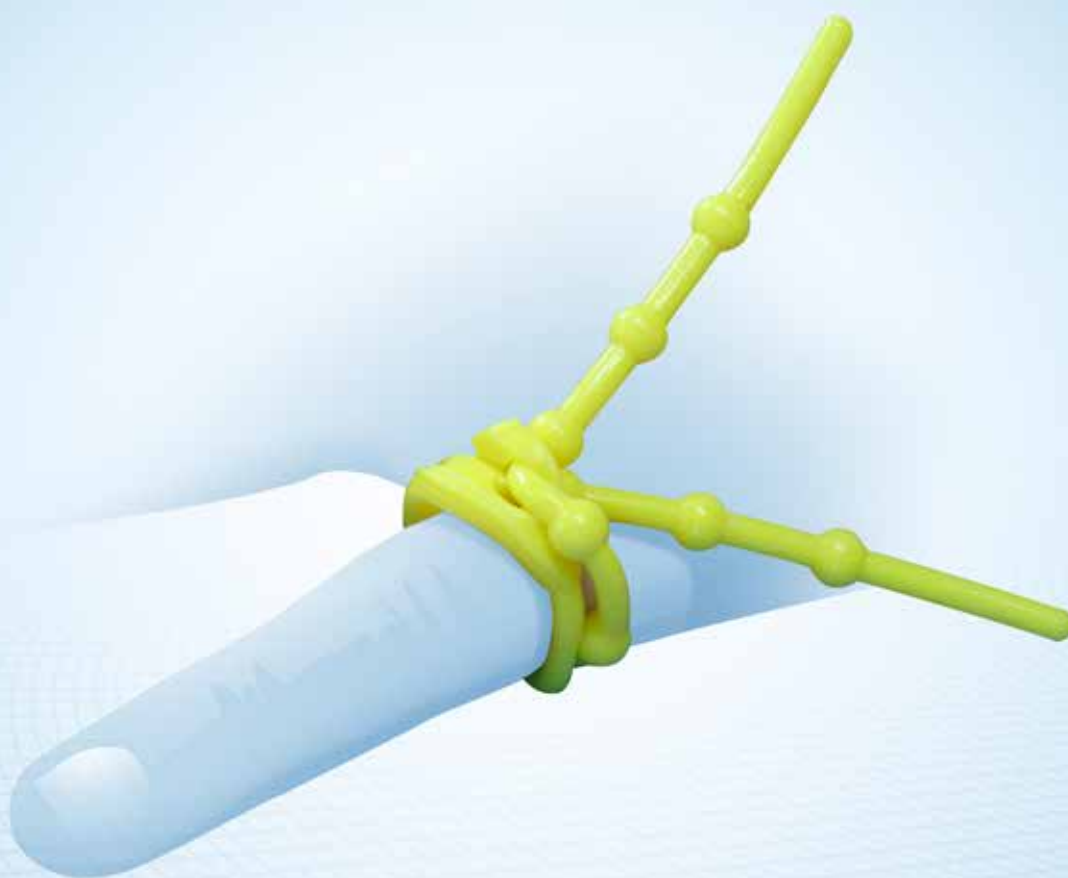

TOURNIKETIX

ref.: 0229 01 01 (yellow) or 0229 01 02 (blue)

TOURNIKETIX

The only specifically designed-for-purpose, reusable finger tourniquet.

- » Reusable
- » One size-fits-all
- » Achieving a totally exsanguinated finger
- » Bright blue or yellow highly visible tails make it totally unforgettable
- » Removable without rolling it over the wound or any risk of cutting skin
- » Environmentally friendly since it is reusable

For optimal use, follow the detailed instructions below. Placement of the Tourniketix is simple, easy and repeatable. It is used under local (LA) or locoregional (LRA) anaesthesia.



1. The Tourniketix is prepared before placement by forming one or two loops, passing one, or both (as preferred), silicone beaded strand(s) through holes in the central crosspiece.
2. The Tourniketix is then placed on the digital pulp and tightened by pulling on the two strands.
3. At this point, the Tourniketix is rolled down towards the base of the finger to obtain exsanguination efficacy.

If during this manoeuvre the Tourniketix slip over skin without wrapping around itself then tourniquet tightness may not be maintained. Tourniketix has to wrap around itself to reach finger base and then to maintain finger vascular tightness during surgery.

4. The surgical procedure (suture, exploration, etc.) may begin.

5. Removing it is easy: simply pull on the central crosspiece to loosen and remove the tourniquet. It can be sterilised and reused.

When a totally bloodless field is not required, the Tourniketix can be used with a single loop: the arm is held above heart level for a few minutes and the It is applied directly to the base of the finger.

NB1 - Do not use Tourniketix on small fingers.

NB2 - Do not use Tourniketix in paediatric cases. These contra-indication are related to Tourniketix dimensional characteristics.

NB3 - Do not clamp Tourniketix with a metallic instrument to maintain this finger tourniquet, this may initiate small cuts into silicone resulting in potential secondary rupture.

TOURNIKETIX^{*}

- ref.: 0229 01 01 (yellow) or 0229 01 02 (blue)


Simple answer to a real issue^{*}: digit tourniquet TOURNIKETIX

Available in yellow or blue

Sold in packs of 10

Reusable

Sold no sterile

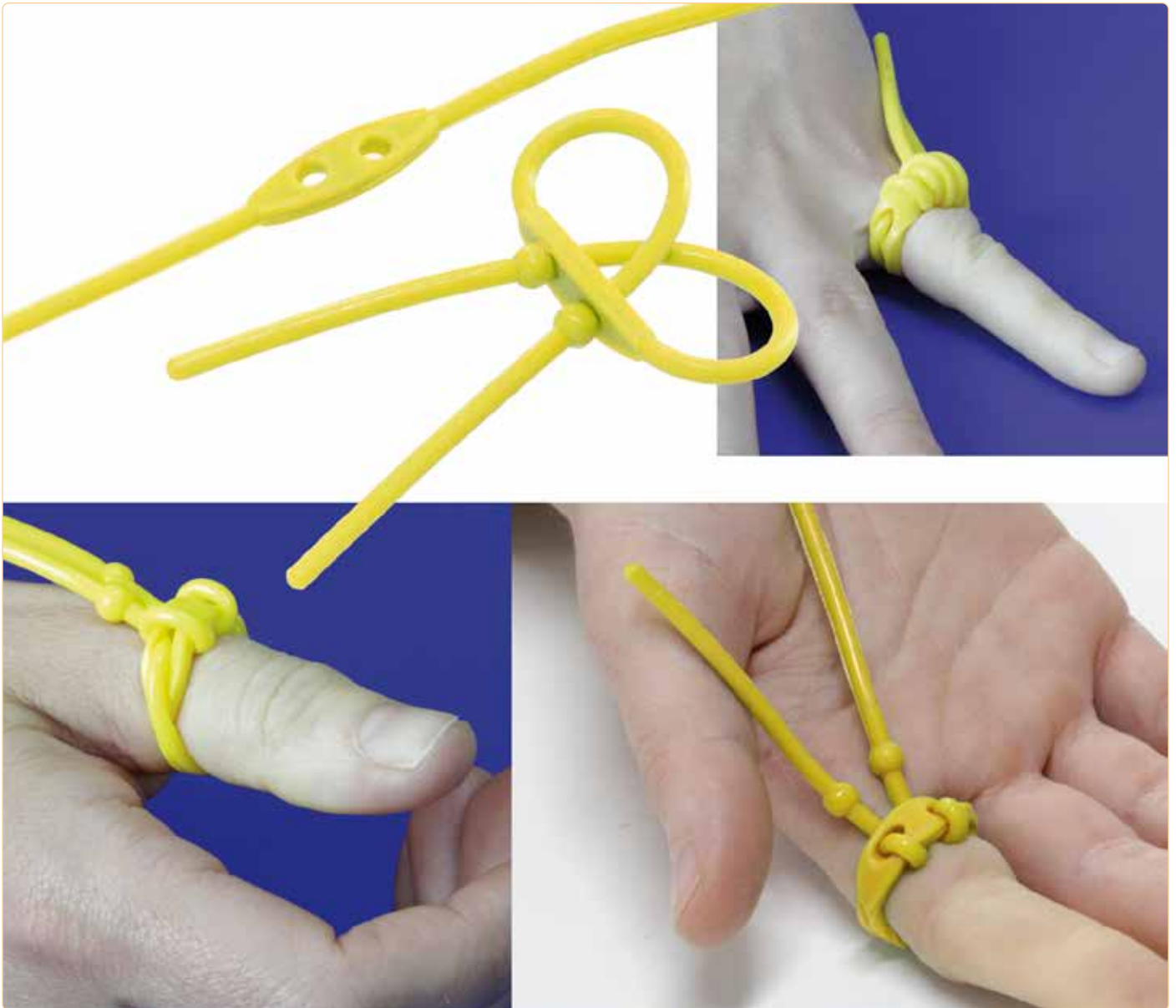

^{*} Long-term functional results of digital ischemia under tourniquet: about 2 observations.

SOFCOT 2010 / Florence Mallard (Angers) Julien Richou, Dominique Le Nen



QUICK VIDEO TO PRESENT EXSANGUINATION

by **Stéphanie Gouzou**
Strasbourg University Hospital

DISTRIBUÉ PAR

Aked
44 C, rue de Bray
F 35510 Cesson-Sévigné
Tél. : +33 (0)2 23 35 53 43
Fax : +33 (0)9 72 26 96 49
Mail : contact@aked.bzh

AKED.bzh